KULLANIM KILAVUZU

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INSTRUCTIONS FOR USE

Bariyer Sprey Instructions for Use

Note: Please read this manual carefully before using the product.

Ingredients:

(1R,2S,5R)-2-isopropyl-5-methylcyclohexanol, sodium hydrogen carbonate, 7-(dimethylamino) phenothiazin-3-yliđene]-dimethylzincium; chloride, 1,2-propanediol, deionized water.

Purpose of Usage

Memethol Barrier Spray, is intended to reduce ea and eliminate the complaints of hemorrhoids and anal fissure patients

Who Can Use

Adults with anal fissures and hemorrhoids problems can use this product. Pregnant and lactating women, children under the age of 15 can not use.

Indications

• MEMETHOL, relieves the pain and itching that the most important symptoms of hemorrhoids with a nonpharmacologic way. It is not drug.

• MEMETHOL, creates a mechanical barrier in the outer layer of the skin so protect the irritated anal area against microbial agents.

• The product can remain in the epidermis by dint of the film layer formation

• During treatment piles get smaller day after day and disappear.
• Ensures compliance with the pH of the anal region
• With using MEMETHOL, fissure and hemorrhoids patient's pain, itching and hygiene problems relieves, everyday life quality is achieved.

Contraindications

• Only used by patients who have been diagnosed as hemorrhoids. Rectal prolapse, anal varicose veins and so on patients should not use.

• Use only for pain from hemorrhoids, anorectal abscess patient's pain may be continue.

• Pregnant and lactating women and children under the age of 15 can not use.

• People who are allergic to any of the ingredients in the product content can not use.

Usage

- 1) Before usage please ensure clean and dry the anal region.
- 2) Remove from the box, shake the bottle.
- 3) Be in a position that hitting the hemorohoidal region for appling.
- 4) Bottle's pie dial-up and spray 7-8 times to the hemorrhoids. Avoid contamination with the genital area.
- 5)After spray, wait for 1-2 minutes in a seated position is recommended.
- 6) After waiting for 1-2 minutes you can wear your underpants
- 7) If you wear your laundry immediately maybe painted blue
- 8) Wash the hands and contaminated elsewhere with soap and water.
- 9) Product should use once daily as described until bottle depleted, implementation without a break is so important.

Frequency of application

• Use the product only once a day.

• In case of healing the complaints before the end of the product bottle, should be continued using once a day until end of the bottle for the treatment in the region.

• The first bottle during use, if the reduction in piles have not reached the desired level, recommend to use a second bottle without interruption. (In severe cases direct two bottles is recommended to use; specially 3. and 4. grade)

Handling Precautions

• Bottle must be shaken before each use.
• It should be used described manner, and amount without interruption every day until end of the bottle.

• Do not use with other products applied to the anal area.

Safety Precautions

- Keep out of the reach of children.
- Do not drink
- Do not contaminate your mouth, nose or anywhere outside of the hemorrhoids.
- In case of contamination your hands, your eyes and your mouth rise immediately with plenty of water.
- Do not contaminate your clothes and your belongings.
- Do not use **incorrectly and Results**
- In case of contamination mouth, nose, eyes maybe burning sensation occurs. Immediately wash with plenty of water.
- In case of failure to comply with the recommended usage or sing more than the spray amount it can cause itching, irritation of the skin, irritation.
- Products should be stored at room temperature.

DO NOT REFRIGERATE, PRODUCT IS PERISHABLE.

Considerations on the First Use

Always read the manual before the first use. Pay attention to warnings before and during application. In case of allergies after your first use

of the product, stop usage

Storage Conditions

- Keep out of the reach of children.
- It should not be exposed to direct light and heat.
- Recommended storage temperature is 15-35°C.
- Store in box and cool the environment.

Do not store in refrigerator.

Should be followed for a permanent cure

- Do not eat hot and spicy food
- Do not drink alcohol .
- Do not force anal region while defecating.
- consume plenty of water for easy defecation
- Stay away from foods known to cause constipation.
- Every day eat one kiwi for comfortable toilet is recommended.
- Please prefer to eat wholemeal bread instead of regular bread.

Side Effects

There are no known adverse effects of the product.

MemeThol Barrier Spray does not include any cortisone and hormone-like derivatives. Skin friendly, was determined by analyses

This manual is prepared according to EN ISO 62366-1:2015, into force on 01.05.2017 was entered.

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